


Preliminary Directions for Use

NeoNatalie™


*Resuscitation Training Simulator for the
“Helping babies breathe” Project ⁽¹⁾*

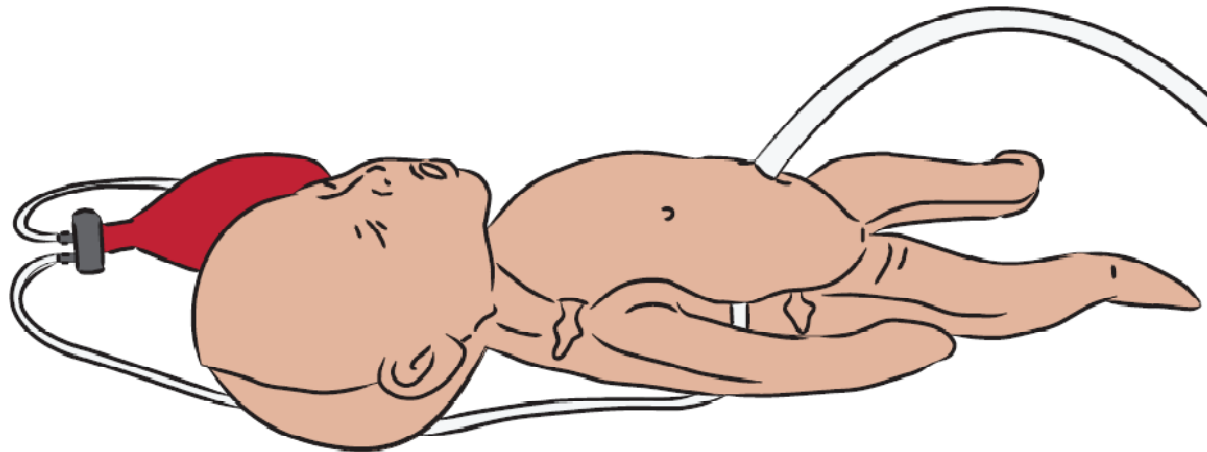

In India we have named this as
Baby Panna



Product overview

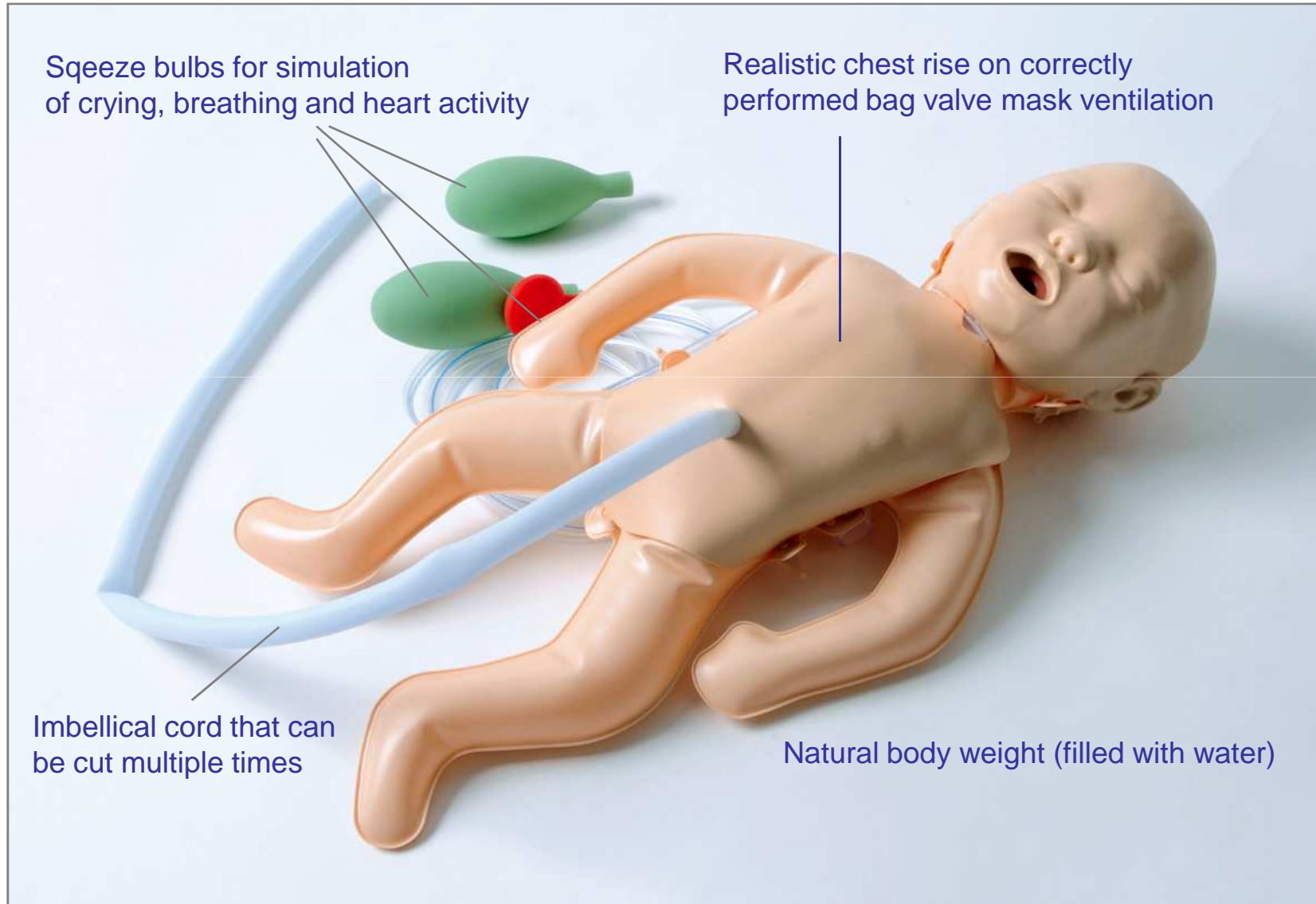


Bag valve mask ventilation

Realistic chest rise when correctly performed bagging
No chest rise when the neck is flexed or hyperextended



Simulation

The instructor can simulate crying, spontaneous breathing and present pulse, for very efficient and realistic training in patient scenarios

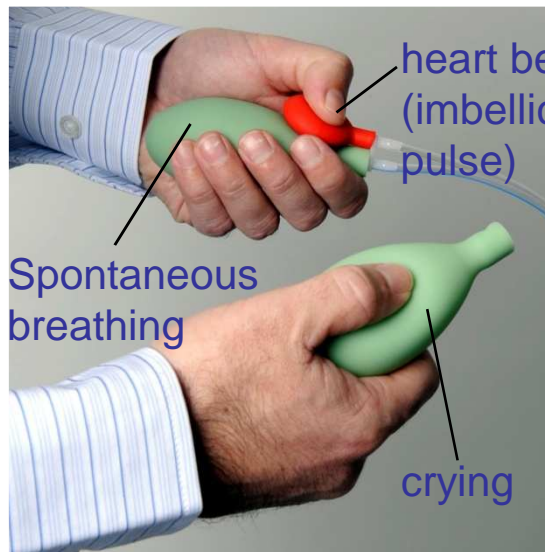

Fill with water, for extra realism



Unscrew filling cap.
Fill from tap or suitable
container.



Reinsert filling cap
before opening the
air valve.



Additionally, blow in
small amounts of air
so NeoNatalie™
becomes firm.



Close air valve and hide
filling extension under
face skin.



Snap face skin into
place.

-Or fill with air, for immediate readiness




Open air valve.



Blow until Neo Natalie™ feels firm.



Close air valve.

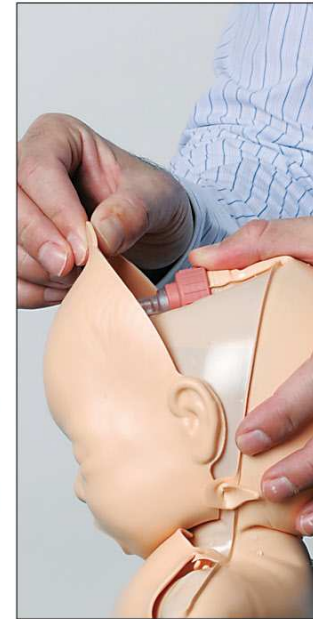

Hide filling extension under face skin.



Snap face skin into place.

Field Test of **NeoNatalie** Neonatal Simulator

- a Training Tool for the IAP NRP FGM Project



Reason for Conducting the Field Test

India is the ***first country*** to use the NeoNatalie neonatal simulator.

IAP has received several hundred units of the new NeoNatalie neonatal simulator ***at no cost*** from the manufacturer ***Laerdal Medical***.

IAP has ***in return*** agreed to conduct a ***field test*** on the ***use*** of this training tool in ***the IAP Neonatal Resuscitation Program First Golden Minute Project*** (NRP FGM).

Feedback from the field test can result in even more efficient training tools for the NRP program in the future.

Practical Information

Where

It is an online survey.

Link to the questionnaire: <http://web.refleks.com/?surveyid=35706>

Who

Questionnaire should be filled out by the instructor

When

Within one week after each course held

Time needed

The questionnaire is short and takes less than 5 minutes to complete

Contact person

Any queries can be directed to Dr. Naveen Thacker via email drnaveenthacker@gmail.com

Questionnaire: Welcome Screen

Field Test of NeoNatalie Newborn Simulator - Microsoft Internet Explorer

File Edit View Favorites Tools Help

Back Forward Stop Refresh Home Search Favorites

Address <http://web.refleks.com/?surveyid=35706> Go Links

Google Søk + - - - - - Bokmerker Finn ABC Kontrollerer Autofyll Logg på

REFLEKS

Field Test of NeoNatalie Newborn Simulator


Field Test of NeoNatalie Neonatal Simulator

Please take 5 minutes and help IAP conduct a field test of the NeoNatalie neonatal simulator.

IAP has received several hundred units of the new NeoNatalie neonatal simulator at no cost from the manufacturer Laerdal Medical and in return agreed to conduct a field test on the use of this training tool in the IAP Neonatal Resuscitation Program First Golden Minute Project (NRP FGM). Your assistance in filling in this form within one week after each course you may be delivering in 2009 is very much appreciated.

When filling in your response, please keep in mind that the NeoNatalie simulator has been developed to be an affordable simulator (expected cost in India USD 50 + import duty) intended for the basic NRP course, and not for an advanced course including instruction in endotracheal intubation.

If you have any questions to this questionnaire, please contact Dr. Naveen Thacker by email dnaveenthacker@gmail.com.



Next

Questionnaire: Screen 1

Field Test of NeoNatalie Newborn Simulator - Microsoft Internet Explorer

File Edit View Favorites Tools Help

Back Search Favorites

Address <http://web.refleks.com/index.php?surveyid=35706&refc=&httppreferer=http%3A%2F%2Fweb.refleks.com%2F%3Fsurveyid%3F> Go Links

Google Søk Bokmerker Finn Logg på

REFLEKS

Field Test of NeoNatalie Newborn Simulator

===

Background questions

Please indicate your age

20-29

30-39

40-49

50-59

60-69

70+

Please indicate your sex

Female

Male

What is your professional background?

Pediatrician

Obstetrician/other physician

Midwives

Other, please specify:

Comment:

In how many Neonatal Resuscitation Program (NRP) courses have you been an instructor over the last twelve months?

Only this one

2-3 courses

More than 3 courses

In how many courses (including this last course) have you used NeoNatalie to teach the NRP course?

1

2

3

More than 3

Next

Questionnaire: Screen 2

Field Test of NeoNatalie Newborn Simulator

REFLEKS

Field Test of NeoNatalie Newborn Simulator

Questions relating to your last NRP course

Where was the course held?
Please select one

When in 2009 was the course held?
Please select one

How many students participated in this last course?

What was the professional background of the students (tick one or more)

Pediatricians

Obstetricians/other physicians

Midwives

Other nurses

Don't know /NA

Which topics were taught in this last course?

Basic NRP

Advanced NRP, including intubation and other advanced topics

How many NeoNatalie manikins were used?

1

2

3

More than 3, please specify how many:

Comment:

Field Test of NeoNatalie Newborn Simulator

How did you prepare the NeoNatalie manikin for the course?

By filling the body with air

By filling the body with water

IF you filled NeoNatalie with WATER: Did you use the Body Volume Expander (the internal inflatable air bag in plastic)?

Yes

No

Did you experience any problems with the use of the NeoNatalie manikin?

No

Yes; please comment:

Comment:

At the cost of USD 50 + duty, and intended for use in the basic NRP course, how would you rate the value of NeoNatalie? Use a scale from 1 to 6, where 1 is "not valuable at all" and 6 "very valuable".

	1	2	3	4	5	6	Don't know /NA
	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>

On a scale from 1 to 6, where 1 is "not valuable at all" and 6 is "very valuable" - how would you rate each of the following simulation features of the manikin?

	1	2	3	4	5	6	Don't know /NA
Crying	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Breathing	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Pulse	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Cord clamping	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>

Do you have any suggestions for improvements of NeoNatalie, or other comments?

Submit

Questionnaire: Last page

Field Test of NeoNatalie Newborn Simulator - Microsoft Internet Explorer

File Edit View Favorites Tools Help

Back Forward Stop Home Search Favorites Refresh Print Mail Stop

Address http://web.refleks.com/index.php Go Links


Google Søk + - - - - - Finn >> Logg på

REFLEKS

Field Test of NeoNatalie Newborn Simulator

Thank you for your feedback!

We would very much appreciate to have your feedback also after you have held your next course. Your feedback is very valuable to us to make sure that we meet your needs and to help making the NRP course even better in the future!



The image shows a NeoNatalie newborn simulator, a realistic baby mannequin used for medical training. It is lying on its back with a green and red bulb connected to its chest. Behind the simulator is its packaging box, which features the NeoNatalie logo and a photograph of healthcare professionals using the simulator.

Close